



Positioning.  
Skin Protection.  
Stability.  
Convenience.  
No Compromise.

The NEW LOW PROFILE QUADTRO SELECT Cushion sets the standard in the industry for overall performance in wheelchair seating. Each and every user will benefit from a customized fit through the simple push of a knob. The revolutionary ISOFLO® Memory Control™ offers shape fitting capabilities while the user is seated, allowing quick and easy, on-demand adjustment to maximize function. With the built-in stability and simplicity of the LOW PROFILE QUADTRO SELECT Cushion, no longer will you have to sacrifice maximum skin protection to get stability, positioning or convenience. The LOW PROFILE QUADTRO SELECT Cushion does it all. And with no compromise.

NEW! THE ROHO® LOW PROFILE® QUADTRO® SELECT® CUSHION


# LOW PROFILE® QUADTRO® SELECT® CUSHION

## POSITIONING .

The LOW PROFILE QUADTRO SELECT Cushion allows for quick, simple position selection for every user while seated. By pushing the knob to the open position, you can adjust the user's posture to the desired sitting position. By pushing the knob to the closed position, you can maintain this desired sitting posture for maximum function.


## SKIN PROTECTION .

Based on thirty years of research and client experience, our patented ROHO DRY FLOATATION® technology provides unparalleled performance in facilitating blood flow by fitting, matching and tracking the shape of a person.

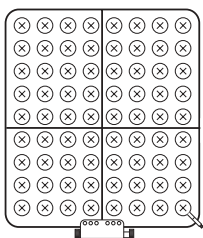

## STABILITY .

The LOW PROFILE QUADTRO SELECT Cushion offers a new level of stability never before found in an air flotation cushion. Stability is enhanced by locking the air into place in each of the four quadrants through the simple use of the ISOFLO Memory Control, minimizing side to side or front to back motion.

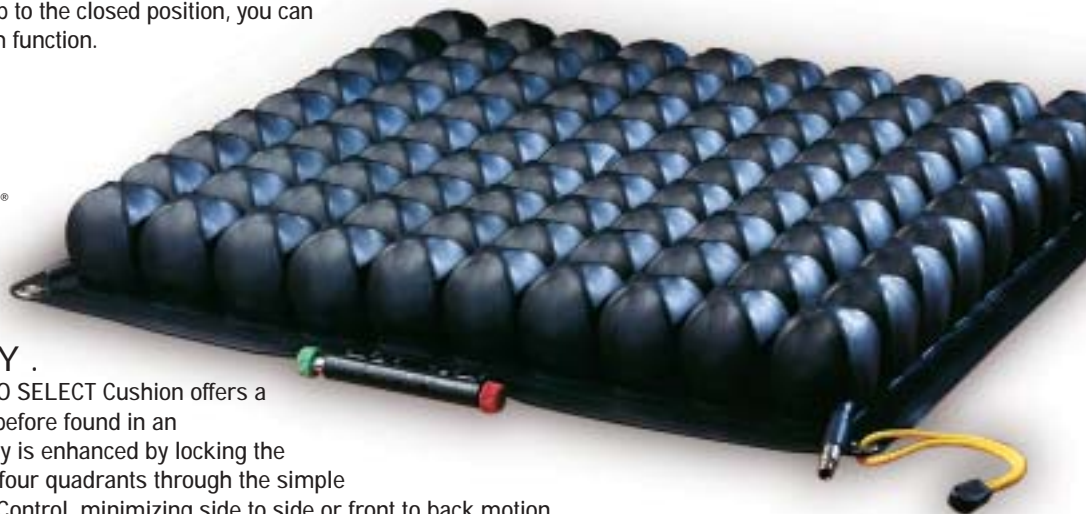

## CONVENIENCE .

All postural adjustments are made while the user is seated, eliminating the need for transfer on and off the chair during the evaluation process. In addition, the LOW PROFILE QUADTRO SELECT Cushion allows the user to selectively change their position throughout the day as their particular activity demands.

## FEATURES & BENEFITS

- ADJUSTABLE:** Fits specific seating needs and unique body shapes.
- LIGHTWEIGHT:** Easy to handle and carry, at just 3.3 lbs./1.4 kg.
- WASHABLE:** Cushion can be easily cleaned with soap & water. Cover is machine washable.
- REPAIRABLE:** Neoprene rubber cushion can be patched or repaired to extend use.
- VENTILATED:** Separate air cells allow air circulation to reach the surface of the skin and provide a cooling environment.

## APPLICATIONS

The LOW PROFILE QUADTRO SELECT Cushion is most effective for users:

- who need a moderate amount of postural support
- with asymmetrical postures
- who are at risk for skin breakdown
- who foot propel
- with partial to no sensation

## SPECIFICATIONS

- INCLUDES:** LOW PROFILE QUADTRO SELECT Cushion, two-way stretch cover, hand inflation pump, repair kit, and operating instructions.
- CONSTRUCTION:** Flame Resistant. Neoprene Rubber.
- SIZES:** Wide range, to fit chair widths 12" to 20"/30 cm to 51cm
- WARRANTY:** 24-Month Limited

## MORE PRODUCTS FROM THE ROHO GROUP



nexus® SPIRIT® Cushion



ENHANCER® Cushion



ROHO® DRY FLOATATION® Mattress



SYMMETRIX® Back System



100 North Florida Avenue • Belleville, IL 62221-5429 • USA • [www.therohogroup.com](http://www.therohogroup.com)  
U.S. & Canada: 1-800-851-3449 • 1-618-277-9173 • fax 1-888-551-3449  
Outside the U.S. & Canada • 1-618-277-9150 • fax 1-618-277-6518