



the ROHO group

shape fitting technology®



STABILITY.

The new CONTOUR SELECT™ Cushion provides the skin protection you expect from a ROHO Group product, with the positioning and stability needed for increased function. The contoured design combined with the simple operation of the ISOFLO® Memory Control™ Unit provides a new level of postural control and stability that has never been achieved in an air cushion.

POSITIONING.

The contoured design positions the pelvis back in the wheelchair, and centers the user comfortably in the middle of cushion. The ISOFLO Memory Control Unit locks air into each of the four quadrants, providing stability by minimizing side-to-side and front to back motion. These features provide the stability that will allow the user to maximize function while minimizing tissue deformation.

SKIN PROTECTION.

The SELECT Family of Cushioning Products comes with a 36-month limited warranty, continuing the ROHO tradition of offering the highest quality and workmanship available in wheelchair seating.

SIMPLICITY.

The CONTOUR SELECT gives you positioning, stability, skin protection and peace of mind, in one simple package.

NEW! THE ROHO® CONTOUR SELECT™ CUSHION

CONTOUR SELECT CUSHION

FUNCTION

Your client needs function? We have positioning and stability for that. The positioning possibilities are obvious when you lay your eyes on the CONTOUR SELECT. The combination of the contoured design and the patented ISOFLO Memory Control Unit provides the unprecedented ability to promote postural stability, maximize function and maintain skin integrity.

SIMPLICITY

Simplicity you can see. Versatility you should expect. Part of what makes it simple is the contoured surface. The CONTOUR SELECT allows the user to sit in the same position and with the same support each time they transfer into the chair. If postural adjustments are needed, they are made while the user is seated – no need to transfer on and off during the process.

SKIN PROTECTION

Don't put your client's skin at risk. Thirty years of research, experience and refinement give you the peace of mind that you are providing your client with unparalleled protection with the selection of a ROHO seat cushion. There is little question – if you teach your client proper use and care of their ROHO cushion, you will have given them a powerful advantage against skin integrity issues and the associated health concerns.



We highly recommend distribution through a certified rehab professional when ordering and fitting ROHO Group products.

FEATURES & BENEFITS

- ADJUSTABLE:** Fits, matches and tracks client shape to manage integrity, positioning and stability.
- LIGHTWEIGHT:** Adds little weight to entire mobility system.
- WASHABLE:** Cushion can be easily cleaned with soap and water. Cover is machine washable.
- REPAIRABLE:** Neoprene rubber cushion can be patched or repaired to extend use of cushion.
- VENTILATED:** Separate air cells allow air circulation to reach the surface of the skin and provide cooling environment.

APPLICATIONS

The CONTOUR SELECT Cushion is most effective for users who:

- require a significant amount of postural support
- have poor to fair seated balance
- are at moderate to high risk for skin breakdown
- have symmetrical postures, or minimally asymmetrical postures
- sit in tilt and/or recline systems
- have conditions that will change over time

SPECIFICATIONS

- INCLUDES:** CONTOUR SELECT Cushion, two-way stretch cover, instruction booklet, hand inflation pump, repair kit, and operating instructions
- CONSTRUCTION:** Flame Resistant, Neoprene Rubber
- SIZES:** Wide Range, to fit chair widths 15" to 20"/38cm to 51cm
- WEIGHT:** Varies by size, approximately 3.6 Lbs./1.6 Kg (based on size CS99C)
- WEIGHT LIMIT:** Unlimited (Cushion must be properly sized to client)
- WARRANTY:** 36- Month Limited Warranty
- MEDICARE PART B CODE:** K0108 Adjustable Skin Protection and Positioning Wheelchair Cushion

The ROHO Group, Inc. has a policy of continuous product improvement and reserves the right to amend specifications presented in this brochure.

MORE PRODUCTS FROM THE ROHO GROUP



HIGH PROFILE® QUADTRO® SELECT®
For skin protection and more involved asymmetrical postures

nexus® SPIRIT®
A lightweight cushion, for skin protection and stability



100 North Florida Avenue • Belleville, IL 62221-5429 • USA • www.therohogroup.com
U.S. & Canada: 1-800-851-3449 • 1-618-277-9173 • fax 1-888-551-3449
Outside the U.S. & Canada • 1-618-277-9150 • fax 1-618-277-6518