



SKIN PROTECTION.

SHAPE FITTING.

**PROVEN
PERFORMANCE.**

LIGHTWEIGHT.

For over 30 years, the ROHO HIGH PROFILE and LOW PROFILE Single Valve cushions have continued to be the market leaders in skin protection seat cushions - they are the benchmark by which all other skin protection seat cushions are compared. The HIGH PROFILE and LOW PROFILE cushions offer an optimal environment for clients with skin integrity issues. The cellular design and the soft pliable material allow the client to be immersed into the cushion for maximum pressure redistribution, while decreasing friction and shear. The simple adjustment allows the cushions to fit the shape of the client, minimizing tissue deformation and maximizing blood flow. No other air filled seat cushion can match the proven performance and quality standards of ROHO's DRY FLOATATION® Technology. Nothing feels like a ROHO!

THE ROHO® HIGH PROFILE® AND LOW PROFILE® CUSHIONS

HIGH PROFILE AND LOW PROFILE CUSHIONS

SKIN PROTECTION

Thirty years of research, experience and refinement of ROHO DRY FLOATATION® technology provides the client with unparalleled skin protection.

SHAPE FITTING

The unique cellular design of the HIGH PROFILE and LOW PROFILE allows the cushions to constantly conform and track movements of the body. The simple adjustment and conformity of the cells accommodates and meets the unique skin integrity needs of each client throughout the day.

PROVEN PERFORMANCE

Laboratory tests prove ROHO will outlast other air filled cushions and maintain their effectiveness over time, even after exposure to contaminants and daily wear and tear.

LIGHTWEIGHT

At only 4.4 and 3.2 lbs, the HIGH PROFILE and LOW PROFILE cushions are easy to transport and handle, which minimizes fatigue of the client.



FEATURES & BENEFITS

- ADJUSTABLE:** Shapes to the individual addressing their specific seating needs.
- LIGHTWEIGHT:** Adds little weight to entire mobility system.
- WASHABLE:** Cushion is easily cleaned with soap and water. Cover is machine washable.
- REPAIRABLE:** Neoprene rubber cushion can be patched or repaired to extend use.

*Also available in Dual Compartment, for front to back or side to side positioning for clients with asymmetrical postures.

SPECIFICATIONS

- INCLUDES:** HIGH PROFILE or LOW PROFILE Cushion, cover, hand inflation pump, repair kit and operation instructions.
- CONSTRUCTION:** Flame Resistant, Neoprene Rubber
- SIZES:** Wide Range, to fit chair widths 12" to 28"/30cm to 71cm
- WEIGHT:** Varies by size, approximately 4.4 lbs./2.0 kg. for HIGH PROFILE, 3.2 lbs./1.5 Kg. for LOW PROFILE (based on 9x9 sizes)
- WEIGHT LIMIT:** Unlimited¹
- WARRANTY:** 24- Month Limited
- MEDICARE PART B CODE:** K0108, Adjustable skin protection Wheelchair Cushion

*DUAL COMPARTMENT MEDICARE PART B CODE: K0108, Adjustable skin protection and positioning Wheelchair Cushion

¹ Cushion must be properly sized to client

APPLICATIONS

The HIGH PROFILE Cushion is recommended for clients who:

- Are at high risk for skin breakdown
- Have no sensation
- Have current impaired skin
- Have history of skin breakdown
- Need assistance with weight shifts and transfers

The LOW PROFILE Cushion is recommended for clients who:

- Are at moderate risk to high risk for skin breakdown
- Have no or partial sensation
- Are able to do independent weight shifts
- Propel their wheelchair with their feet

MORE PRODUCTS FROM THE ROHO GROUP



CONTOUR SELECT™ when additional stability is also desired.



HIGH PROFILE® QUADRO® SELECT® for skin protection and more involved asymmetrical postures



The ROHO Group, Inc. has a policy of continuous product improvement and reserves the right to amend specifications presented in this sell sheet.



100 North Florida Avenue • Belleville, IL 62221-5429 • USA • www.therohogroup.com
U.S. & Canada: 1-800-851-3449 • 1-618-277-9173 • fax 1-888-551-3449
Outside the U.S. & Canada • 1-618-277-9150 • fax 1-618-277-6518